



# Mobile manual

Version: September 18th 2020



# Reducept mobile app

The Reducept mobile app allows you to play Reducept without the Oculus Go headset. In this manual you install Reducept on your smartphone and link your game to the subscription you received from your practitioner.


## Content

<b>Installing Reducept on your phone</b>	<b>3</b>
<b>Login to the Reducept app</b>	<b>6</b>
<b>Play the Reducept game in VR</b>	<b>8</b>




# Installing Reducept on your phone


1. Open the iTunes Store (on iPhone) or the Google Play store (on Android phones).

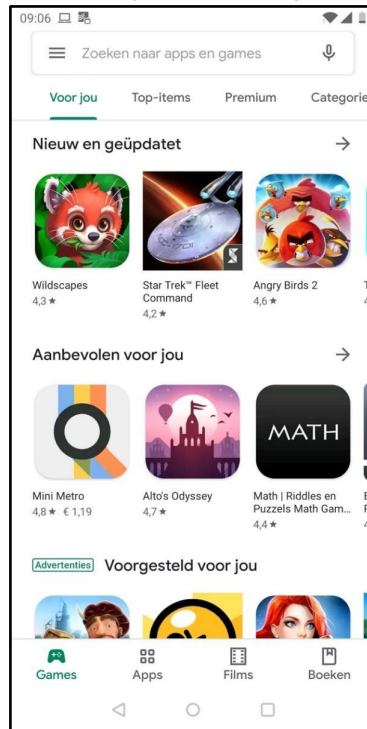

2. Search in the search field for 'reducept'. Choose the Reducept app in the search results.

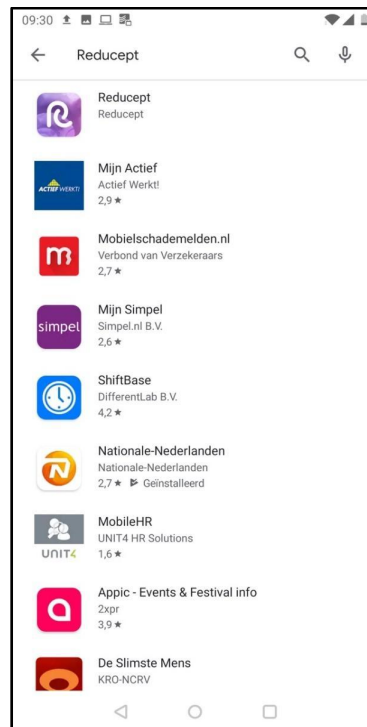

3. Press 'Install'.

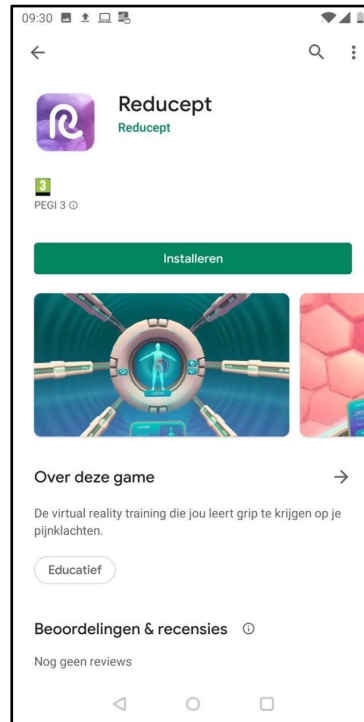

4. Press 'Open'. The Reducept app has also been added to your phone's home screen




## Login to the Reducept app


1. Start the Reducept app (see the step 'Install Reducept on your phone').



2. Press the arrow in the 'User' menu.

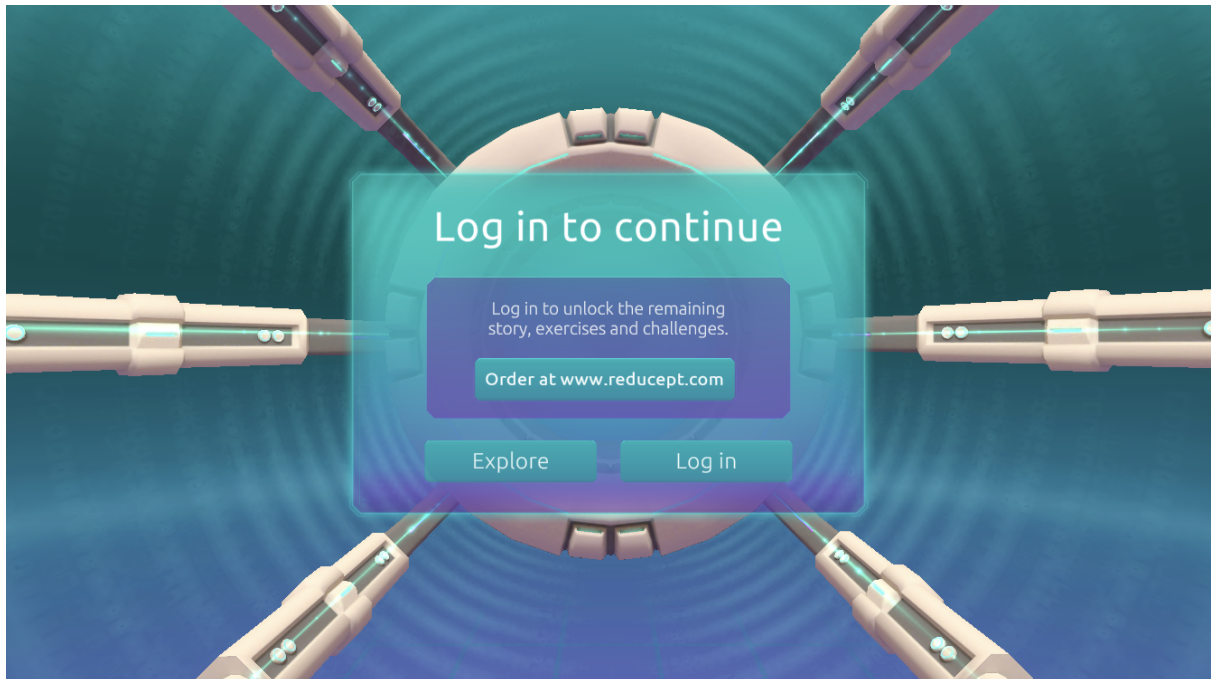

3. Enter your email address and the password you have set in the Reducept dashboard and then press the 'Login' button.

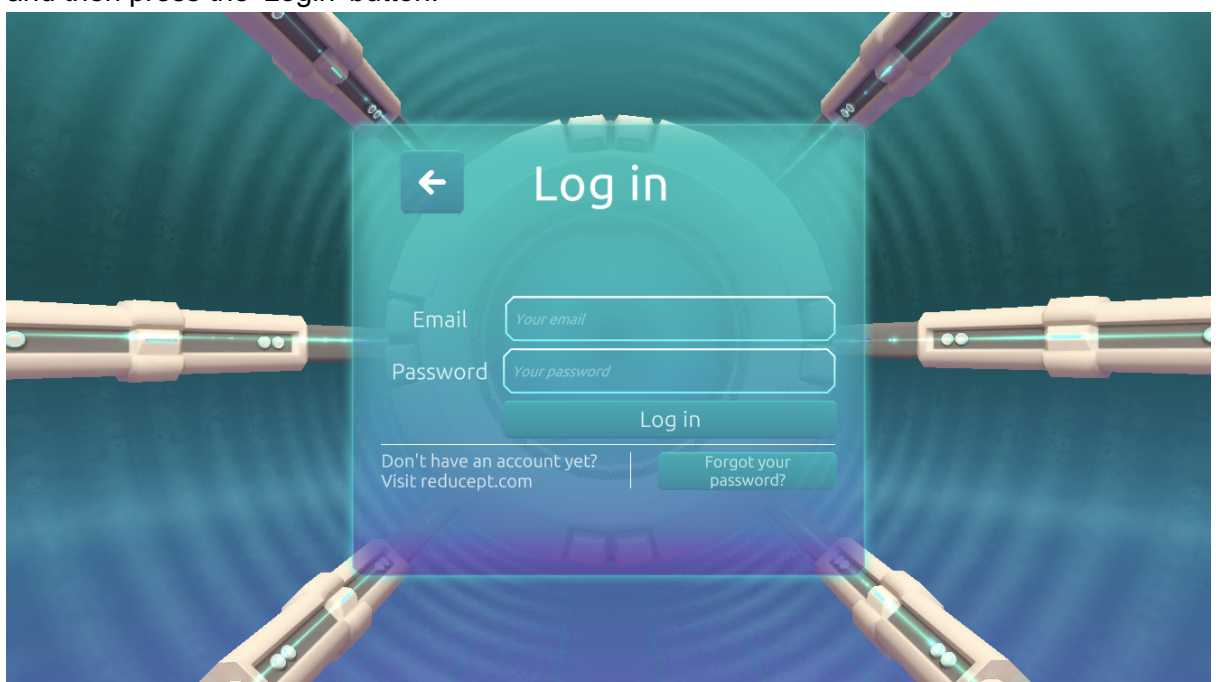


4. You're logged in now.




## Play the Reducept game in VR

1. Start the Reducept app (see the step 'Install Reducept on your phone'). Make sure you are logged in (see the step 'Login to the Reducept app').
2. Open the 'Options' screen by pressing the cogwheel in the upper right corner.

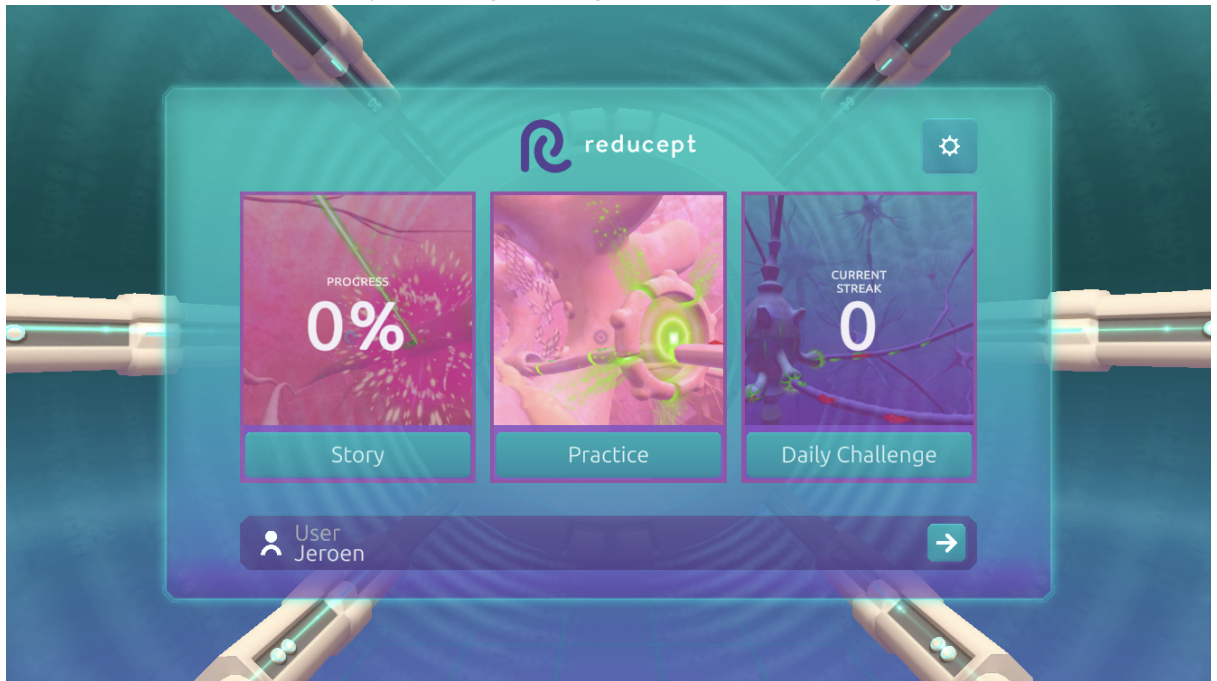

3. In the 'Options' screen, press the 'Switch VR' button.

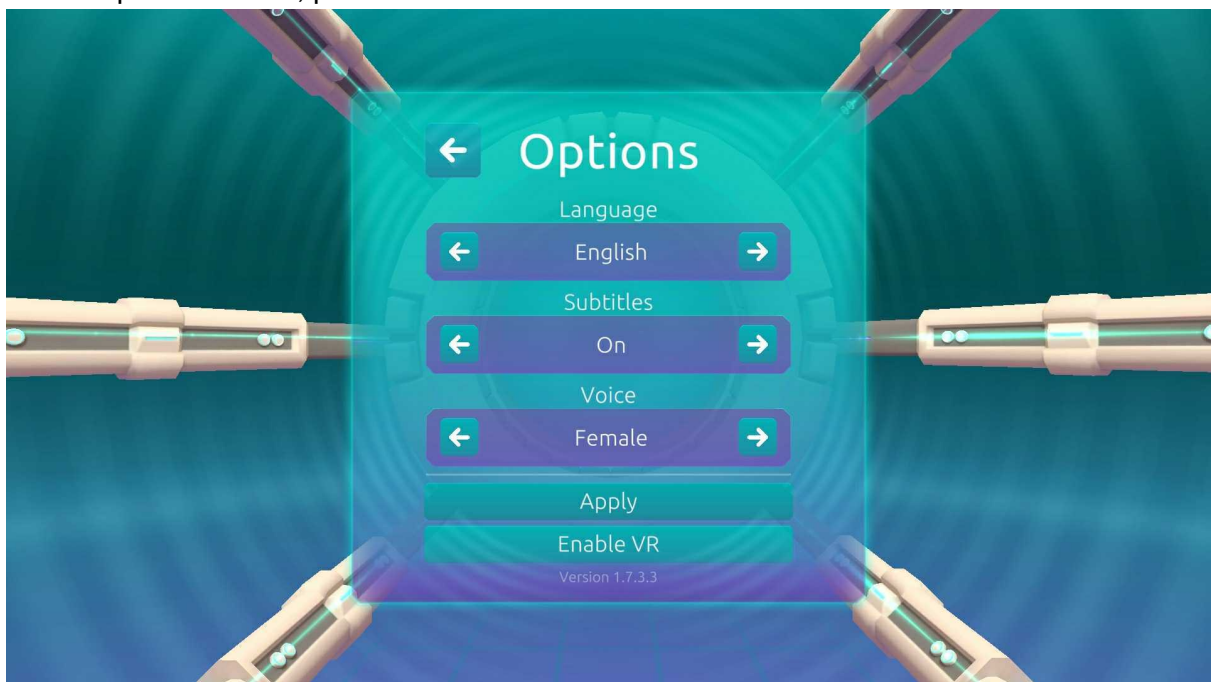



4. **(OPTIONAL)** Put your phone in a dedicated VR headset and place the headset on your head.

